

10/07/2020

## Smart Pump Programming Primary Infusion

Impacted Areas: All Nursing Areas/Roles Utilizing BBraun Pumps

### Reminder/Refresher

#### When *starting* a new Primary Infusion:

- Power on the pump and wait up to 2 minutes for the pump to connect  
OR
- Take the pump out of stand by
- **YOU MUST SMART PUMP PROGRAM FROM THE BBraun LANDING SCREEN**

#### B Braun Landing Screen



(ex. starting a new IV fluid, switching to a different IV fluid, between intermittent medications hung as a Primary)

#### When a Primary infusion is *complete*:

- CLEAR THE PUMP
- Place in Stand By or Power Off